



Scanner Sleeve

Autoclavable sleeve.




Scanner Glass

Please use gloves to avoid finger print marks on glass.



Step 1:

After sterilization, clean glass with soft cloth until clear.

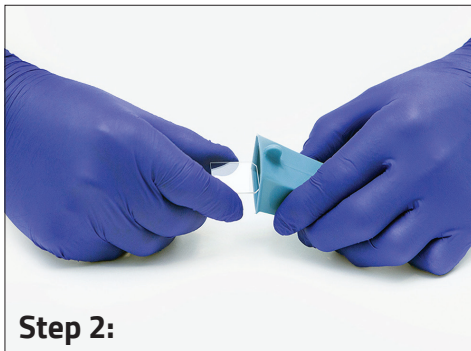

Step 2:

Slide glass with beveled end pointing forward into sleeve.



Step 3:

Confirm glass is placed all the way to the end of sleeve.



Step 4:

Place sleeve onto wand.



Step 5:

Confirm sleeve is pushed all the way on and flush with wand.



Sleeve should look like this. Now you are ready to scan.



Pull sleeve off and remove glass. Sterilize.