

## MagiCalibrator Sterilization Instructions

---

To ensure that you get the best results and longest life out of your new sterilizable MagiCalibrator, please follow the instructions below:

### **How to Sterilize**

The product can be manually cleaned with antimicrobial soap and water. To clean the MagiCalibrator, ultrasonic cleaning with enzymatic or non-enzymatic solutions that facilitate cleaning may be used. Dry the MagiCalibrator and then put into an autoclave pouch.

Autoclave pouches are recommended to keep the MagiCalibrator sterile until needed. Autoclaving is the suggested method of sterilization. Follow your sterilizer manufacturer's recommendation for times and temperatures. For example, the CDC recommendations for steam sterilizers is 250-270°F for 30 minutes, and heat sterilizers 320-340°F for 1 hour. DO NOT use chemical sterilization techniques for the MagiCalibrator. Store the sterilized MagiCalibrator in a dry and dust free location. Keep the product sealed in the sterilization pouch until ready to use. Sterile area to be assembled. Using clean gloves, inspect the glass insert for any signs of spotting from the sterilization process. Use an alcohol pad to remove any stains much like you would a mouth mirror. Scan speeds are optimal with a clear glass insert. Follow the instructions that came with the product for placing the insert into the sleeve and the sleeve on the wand. Place a suitable barrier over the sleeve if not immediately scanning.

