



**MUST BE READ
PRIOR TO USING
THE GOST
SYSTEM**

Dear Doctor,

Thank you for ordering our exclusive GOST System.

Instructions For Use:

Prior to using our GOST Discs, please use our Guided Magic Strip **BCMS** (Purple Single Sided-0.075mm) to break contact followed by **MSG010DS** (Yellow Double Sided-0.10mm) to create enough space for the guided diamond free area to insert easily. **Make sure to tighten the strip using the colored knob.**

It is very important to open a gap of 0.10 mm between the teeth to allow the guided disc to run freely without getting stuck, **which could cause the disc to bend.**

While using the GOST kit, please use the following steps:

1. Use the yellow guided disc **GOST0.15** and then continue moving up in size using the red **GOST0.20**, blue **GOST0.30**, green **GOST0.40** and black **GOST0.50** (If needed). Prior to moving up in size make sure the discs run smoothly until no resistance is felt.
2. It is a must to use the **disc guard (MDISC-GUARD1)** around the disc to create extra protection to the patient.
3. **STOP! PLEASE NOTE:** When using our discs make sure to place the 4-1 Handpiece in the locked position. Not locking the disc in to position correctly will cause the handpiece and disc to become damaged and will void its warranty. The best way to check is to tug the disc with the guard prior to turning it on, to make sure it is in there snug.
4. When utilizing the 4-1 Straight Handpiece, make sure to set the RPM knob on the GOSTUNIT between 5-8. Which will equal 5,000 to 8,000RPM.

Thank you from all of us at Strauss Diamond, and we hope that you enjoy our products!