



Instructions for Use – Magic Strips

General instructions

- For the safety of user and patient, it is highly recommended that only qualified dentists should work with the Magic Strip.
- Strauss & Co.'s magic strips designed for several uses and are autoclavable.
- The strips are designed to work manually in the interproximal gap.
- Avoid touching soft tissue with the strips.
- Sterilize the magic strip before use.

Sterilization instructions

Instructions for cleaning and sterilizing the magic strip

Disinfect and sterilize the strips before every use.

Cleaning:

- Clean the strips by dipping them in disinfection solution that contains a corrosion inhibitor for a minimum of 10 minutes.
- If needed -use brush to clean thoroughly.
- Rinse several times to remove all the disinfection solution.

Cleaning with Ultrasound:

- Strips may be ultrasonically cleaned.
- A cycle of 5 minutes is recommended, using a general-purpose cleaner.
- Rinse several times.

Sterilizing magic strip–autoclave:

- Place the strip in sterilization bags.
- Autoclave the strip for a minimum of 10 minutes at a temperature of 135°C.

Following sterilizing or cleaning methods, dry the strips and store in a clean and moisture free environment.

Note:

- Use sterilizing and cleaning devices and materials according to the manufacturers recommended procedure.
- It is the responsibility of the user to ensure that sterilization is effective.