

## 2. Surface Cleaning

Clean with brush and wipe off by cloth. Lubrication should be done every time after surface cleaning. It cleans better to apply a little amount of alcohol on the cloth.

⚠ Do not place handpiece into any chemical liquid such as disinfectant.


## 3. Inside Cleaning

Take the same steps with the process of spraying lubricate oil. Run the handpiece at low speed every after lubrication. Repeat the process at least twice, to drive off any dirt. Do inside cleaning once a day.


## 4. Cleaning Internal Gears

Any foreign substance such as saliva, blood, chemical, inside the head needs to be immediately removed by the steps followed.

- ① Disassemble the head (Fig.4).
- ② Use brush or cloth to thoroughly clean each part (Fig.4).
- ③ After aligning position (Fig.5), Head needs to be securely screwed on sheath.
- ④ Lubricate every time after cleaning.

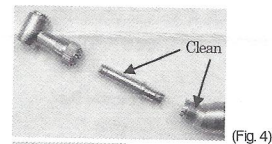

(Fig. 4)

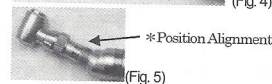

(Fig. 5)

⚠ Do not place handpiece into any chemical liquid such as disinfectant.

## STERILIZATION

⚠ Do not put Handpiece through ultrasonic disinfection solution cleaning procedure. Do not use cold sterilization or dry heat.

⚠ Please use Class B sterilizing device stated in EN13060.

1. Clean with brush and wipe off by cloth. Refer to the section, Maintenance, 2. Surface Cleaning.
  2. Lubricate with Spray. Refer to the section, Maintenance, 1. Lubrication.
- ⚠ Blood, debris, other substance left inside Handpiece, can be solidified during sterilization and cause break-down. Please clean and lubricate well before autoclave.
3. Place Handpiece in sterilization pouch. Autoclave at temperature 134-137°C(2Mpa) for 15min.
  - ⚠ Prevent the bur on handpiece from hitting others in time of autoclave.
  - ⚠ Please operate Autoclave dry cycle at temperature 135°C.
  4. Leave Handpiece in the sterilization pouch stand till excessive oil inside flows out.

## REPLACEMENT PARTS

### 1. Replacement Head

MODEL No.	Replacement Head Item #
EC-20L, EG-20L	EH-20L
EC-30BL, EG-30BL	EH-30BL
EC-35BL, EG-35BL	EH-35BL
EC-30BLP, EG-30BLP	EH-30BLP
CTM-P01, CGM-P01	EHM-P01

MODEL No.	Replacement Head Item #
ECN-20L, EGN-20L	EHN-20L
ECN-30BL, EGN-30BL	EHN-30BL
ECN-35BL, EGN-35BL	EHN-35BL
ECN-30BLP, EGN-30BLP	EHN-30BLP
CTN-P01, CGN-P01	EHN-P01

### 2. How to change head

- ① Unscrew pouch nut and remove head. (Fig.6)
- ② Match teeth of head and sheath each (Fig.7), and screw pouch nut tight. Check the head is securely fixed.
- ③ Please change head once every year.

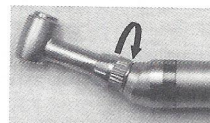


Fig. 6



Fig. 7

## WARRANTY

Trouble, which occurs due to material defect or despite with correct use as the instruction manual states, during the period specified below, is covered under warranty by the manufacturer. Quality deterioration caused by drop of CA Handpiece will be excluded from warranty coverage.

Contra Angle Handpiece: 6 months / Spare Head: 3 months



Advena Ltd. Tower Business Centre, 2nd Flr.,  
Tower Street, Swatar, BKR 4013 Malta



NAKAMURA DENTAL MFG. CO., LTD.  
59-2 Minamicho, Itabashiku, Tokyo 173-0027, Japan  
<http://www.nd-nakamuradental.com>  
TEL: 81-3-3955-5307 FAX: 81-3-3955-5309