

Maintenance:

△ To keep ST Handpiece under good condition, lubricate oil specifically made for dental Handpiece should be used. Use the spray as instructed by its manufacturer.

1. Lubrication

· Before autoclave	: Every time	· After Cleaning	: Every time	· After use	: Every time
--------------------	--------------	------------------	--------------	-------------	--------------

Be sure to lubricate every time after use, before autoclave, or in case that water and/ or dust gets inside. Spray lubrication could also make inside cleaning. It should be conducted twice each time. Please spray lubrication oil for more than 2 seconds.

△ When water goes inside the ST Handpiece, lubricate more than three times.

△ Without being held firm by hand, ST Handpiece could be pushed away by air-pressure of spray. Accidental drop could cause damage on the device.

2. Surface Cleaning

Clean with brush and wipe off by cloth. Lubrication should be done every time after surface cleaning. Although it has effect to apply a little alcohol on the cloth, excessive amount, by seeping inside ST Handpiece, could take off the lubricate oil on ball-bearing. If so, lubricate again.

3. Inside Cleaning

Take the same steps with the process of spraying lubricate oil. Operate trial run at low speed every after lubrication. In order to drive dirt out of nose, repeat the process at least twice. Do the cleaning once a day.

STERILIZATION

△ Do not put ST Handpiece through ultrasonic disinfection solution cleaning procedure. Do not use cold sterilization or dry heat.

△ Please use Class B sterilizing device stated in EN13060.

1. Clean with brush and wipe off by cloth. Refer to the section, Maintenance, 2. Surface Cleaning.

2. Lubricate with Spray. Refer to the section, Maintenance, 1. Lubrication.

△ Blood, debris, other substance left inside ST Handpiece, can be solidified during sterilization and cause break-down. Please clean and lubricate well before autoclave.

3. Place ST Handpiece in sterilization pouch. Autoclave at temperature 132-135°C(2Mpa) for 15min.

△ Prevent the bur on handpiece from hitting others in time of autoclave.

△ Do not operate Autoclave dry cycle at temperature higher than 135°C.

4. Leave ST Handpiece in the sterilization pouch stand till excessive oil inside flows out.

WARRANTY ~ Straight Handpiece : 1 year

Trouble, which occurs due to material defect or despite with correct use as the instruction manual states, during the period specified above, is covered under warranty by the manufacturer. Quality deterioration caused by drop of ST Handpiece will be excluded from warranty coverage.

△ Please ask the store the product is purchased from, for repair.



Advena Ltd. Tower Business Centre, 2nd Flr.,
Tower Street, Swatar, BKR 4013 Malta



NAKAMURA DENTAL MFG. CO., LTD.
59-2 Minamicho, Itabashiku, Tokyo 173-0027, Japan
<http://www.nd-nakamuradental.com>
TEL: 81-3-3955-5307 FAX: 81-3-3955-5309